

Gengy™ - Portable Intubation Manikin

A simple, easy to carry, demonstration manikin to be used to illustrate the techniques involved in endotracheal intubation.

As part of our continuing efforts to develop “easy to use” intubation and emergency medical care devices, Truphatek® has developed the Gengy™ portable intubation manikin that has been specially designed for demonstrating the unique features of Truphatek’s advanced airway products.

The **Gengy™ intubation manikin** mimics the anatomical structure of the adult human airway and is uniquely appropriate for teaching the techniques for use and for promotion of difficult airway products such as the Truview EVO2 and the Tru-FLEX™ articulating stylet.

The **Gengy™ intubation manikin** is light, easy to carry and quick and convenient to assemble on almost any flat surface, making product demonstrations possible in almost any venue.

The packed **Gengy™ intubation manikin** weighs only 930 grams and fits into a 29.5 x 11 x 16.5cm case.

Assembling the manikin

Place the base (2) on a clean and empty surface. Choose a surface where the clamp (1) can grasp the table top, as illustrated. Open the clamp and insert it through the space in the base until it clamps onto the internal metal bar. Check that the base is firmly held in place.

Insert the connection bars (4) into the connectors (3) and fasten them by tightening the screw fasteners.

Open the jaw locks (5) by loosening the screw provided. Move the upper jaw (maxilla) (6) to adjust the mouth opening, as required and retighten screws to maintain chosen mouth opening.

We suggest choosing a wide opening for first ‘easy’ demonstration or a ‘one-finger opening’ to demonstrate difficult intubations and intubations with “c-spine” injury.

When assembling or disassembling the **Gengy™ intubation manikin**, use only reasonable force sufficient to tighten the screws.

Cleaning instructions

Use a cloth or paper towel moistened with water or alcohol to clean the surface of the manikin.

Replacement of Parts

After every 200 intubation demonstrations, replace the pink plastic anatomical part by using the two screws.

Part Numbers

4173 ‘Gengy™ Intubation Manikin set

417005 Replacement pink plastic anatomical part and the vocal cord stabilizer.

410001 Replacement **OptiShape™ stylet**

Warranty

There is no guarantee on this product. Replacement parts can be purchased according to need.

How to Order

Contact: helpdsk@truphatek.com

Website: www.truphatek.com

Manufactured by:

Truphatek International Ltd.

9 Haomanut Street

P.O.Box 8051,

Netanya, Israel

Tel: +972-9-8851155



Truphatek International Ltd.

9 Haomanut Street, P.O.Box 8051

Netanya 42504, Israel

Tel: +972-9-8851155, Fax: +972-9-8851212

helpdsk@truphatek.com www.truphatek.com