

New Set Up for Menachem.

PREMIER CAMERA (model DM7362) SET UP INSTRUCTION.

Instructions for a preliminary set up of PREMIER CAMERA in order to provide the best performance of the camera.

The set up is as follow:

Push menu while the main batten is on CAMERA position, and you will get the following menu:

CAPTURE		FUNCTION		SET UP	
SCENE MODE	PROGRAM AE	CAPTURE MODE	SINGLE	BEEP	ON
SIZE	640X480	VOICE MEMO	OFF	FR. NO. RESET	-
QUALITY	FINE	LCD BRIGHT	+ 3	AUTO OFF	3 MIN
SHARPNESS	NORMAL	DIGITAL ZOOM	ON	DATE/TIME	DIVERT
SATURATION	NORMAL	PREVIEW	ON	SYS. RESET	-
WHITE BALANCE	AUTO	DATE PRINT	ON	LANUAGE	ENGLISH
MANUAL WB	CANCEL				
ISO	AUTO				
METERING	AVERAGE				

Change the definitions as defined at the above table.

4

Push menu while the main batten is on VIDEO position, Keep original set-up, as defined at the following table:

CAPTURE		FUNCTION		SET UP	
SIZE	640X480	AUDIO MODE	ON	BEEP	ON
QUALITY	STANDARD	LCD BRIGHT	3	FR. NO. RESET	-
COLOR	NORMAL	DIGITAL ZOOM	ON	AUTO OFF	3 MIN
				DATE/TIME	DIVERT
				SYS. RESET	-
				LANUAGE	ENGLISH

Push menu while the main batten is on VIEW PICTURES position, and you will get the following menu:

PLAY		SET UP	
SLIDE SHOW	3SEC	BEEP	ON
DPOF	-	FORMAT	-
PROTECT	-	FR. NO. RESET	-
LCD BRIGHT	3	CARD INFO	-
RESIZE	-	SYSTEM INFO	-
QUALITY	-	LANUAGE	ENGLISH
COPY TO CARD	-		

Change the definitions as defined at the above table.

Prepared by Shiri Soffer
Engineering Department